



RediCup™ | Drugs of Abuse Test Device


1 Collect specimen.

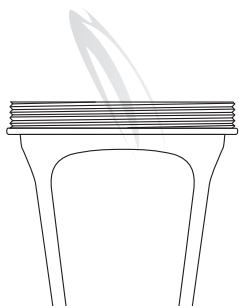


2 Secure cap.
NOTE:
Screw cap tightly to ensure adequate seal.

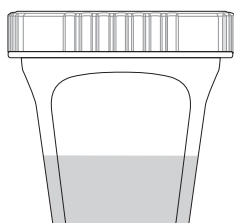


3 Quickly invert cup and place on a flat surface for approximately 30 seconds

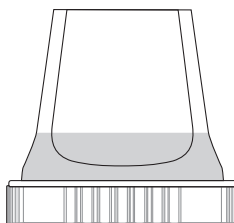

4 Stand upright on a flat surface. If no sample wicking is seen at 1 minute, repeat steps 3 and 4.

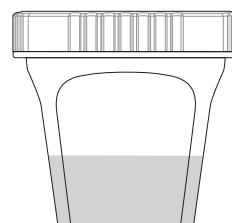

5 Read results at 5 minutes. Do not interpret results after 60 minutes as false results may occur. Note: Any line at all in the test zone is to be interpreted as negative.

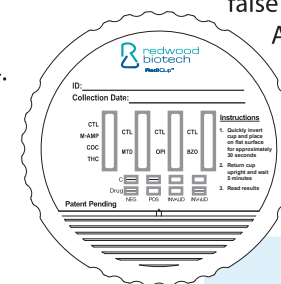

Result Key			
C			
Drug			
	NEG	POS	INVALID

NOTE: ALL POSITIVE RESULTS ARE PRESUMPTIVE AND SHOULD BE CONFIRMED BY AN ALTERNATIVE METHOD (e.g. GC/MS)

Sending Positive* Specimens to Redwood Toxicology Laboratory for GC/MS Confirmation

1. Perform the on-site test according to the instructions provided above.
2. Complete the Preliminary Results Form, indicating the positive and negative test results. **Make sure the donor signs the form.**
3. Secure the positive specimen in the RediCup™. **To prevent leakage, make sure the lid is threaded properly and tightly fastened.**
4. Place the security seal over the top of the RediCup.
5. Fill out the Specimen ID/Chain of Custody Label and indicate the positive drug(s) on the line marked "GC/MS Confirm: _____".
6. Place the label on the RediCup.
7. Instruct the donor to initial the security seal.
8. Seal the RediCup in a plastic baggie with the absorbent material.
9. Make a copy of the Preliminary Results Form and place it in the DHL/Airborne bag (five or more specimens) or the mailer box (fewer than five specimens) with the specimen.
10. Send to RTL for GC/MS confirmation.

***Please do not send negative specimens to RTL for confirmation. Negative specimens will not be confirmed.**